


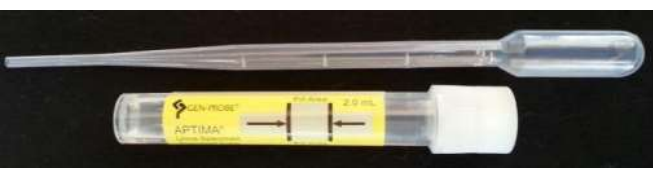





Bacterial Culture Swab with semi-solid transport media	Order number 20211		For Bacterial culture and Gram stain; Candida / Yeast & Bacterial Vaginosis; Group B strep screen
Viral / molecular swab with liquid UTM (Copan flocked NPS swab)	Order number 20403		For NAAT / PCR testing for SARS-CoV-2, Measles, pertussis, Mumps, VZV, HSV, Mpox, Syphilis. Do not use for Chlamydia, Gonorrhoea, or Trichomonas
Aptima swab for Chlamydia, Gonorrhoea, Trichomonas, and M. genitalium	Order number 15066	 The blue swab is the only swab to be placed in the collection tube (the white swab is for cleaning, if required, do not place in the tube)	Females – Cervical swab or vaginal swab for Chlamydia, Gonorrhoea, Trichomonas, M. genitalium; Males – Urethral swab for Chlamydia, Gonorrhoea, M. genitalium
Aptima urine tube for Chlamydia, Gonorrhoea, Trichomonas, and M. genitalium	Order number 15067		First void urine in males (Can be used for testing in females but low sensitivity specimen compared to vaginal swab)
Urine tube for MSU/CSU bacterial culture	Order number 12086		Twist the top of urine tube off, pour in urine sample until it is $\frac{3}{4}$ full, and replace top twisting on
BD SurePath Liquid-based Pap test	-	 Order Number(s): 20265 SurePath Cervix Broom 20266 SurePath Cytobrush & Spatula 20264 SurePath Vial SurePath Vial Base Stand	-
HPV FLOQSwab (Copan 552C)	Order number 15377		Cervical Screening Human Papillomavirus (HPV) Test Do not use for Bacterial culture.

¹ The combined low vaginal swab then rectal swab can **only** be used for Group B Streptococcus culture

² Only one high vaginal swab is required for Candida and Bacterial Vaginosis examination